

Instructions

STEINER Pistol AFTER LOADING TECHNIQUE



STEINER Pistol without adapter

- Brown 1ml Tuberculine Syringe filled with media clicked at Pistol, plunger fixed with screw.
- Luer of any Inner Catheter to be connected with syringe.
- Outer C. inserted by physician at desired position. Commando for loading.
- Loading C.: starting at position "after ET" moving plunger (held between forefinger and thumb) towards 10 μ l or 20 μ l depending how much volume you wish for transfer.
- Handing "loaded pistol" to physician for inserting inner C. after reaching desired position of outer C. Embryologist may use a forceps for inserting inner C.
- Release embryos by slowly moving plunger from Pos. 40 μ l or 20 μ l towards 0

E.g. Dr. Steiner's IVF Center: medium-air-medium with Embryo-air-medium


STEINER Pistol with Adapter

Adapter enables physician to hold both, outer C. and loaded inner C. in one hand for insertion into cervical canal.



STEINER Pistol with adapter

- Brown 1ml Tuberculine Syringe filled with media clicked at Pistol, plunger fixed with screw.
- Insertion of adapter into Pistol (picture below).
- Click outer C. at adapter
- Click inner C. at syringe
- Press white button at left side of Pistol with left forefinger to fix adapter

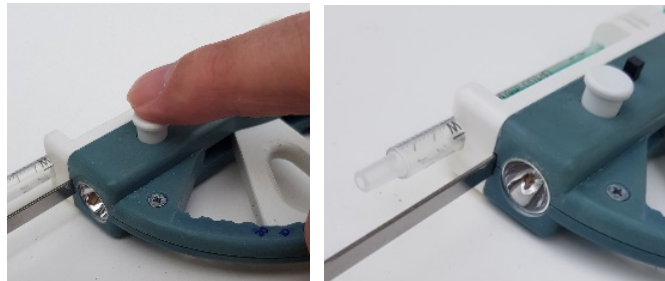

- Adapter inserted above head lamp
- Loading C : See above.
- Press white button at right side of pistol to release adapter for moving adapter plus loaded inner C. 3-4cm into outer C.



- Press left button to fix adapter.
- Hand “loaded pistol” at physician for Embryo Release:
- Holding pistol plus C. in one hand, inserting C. at desired position at Inner Os of Cercical Canal.
- Holding adapter with left hand , pressing right button with right forefinger in order to release adapter. Insert inner C. at desired position.
- Holding plunger of pistol between thumb and forefinger. Move plunger very slowly towards position 0.